



OPERATIONS **MANUAL**

WESTERN CAPE MOBILE CLINIC

(+27) 031 700 1590
admin@savans.co.za
www.savans.co.za

TABLE OF CONTENTS



FRONT OF VEHICLE

- Shore Plug & Cable
- OEM Auxillary Battery



EXTERNAL COMPONENTS

- Connecting Shore Plug
- Fresh Water Storage Refill
- Privacy Door Lock



INTERNAL COMPONENTS

- Internal & External Stadiometer
- Folding Work Station
- Work Station: Hand Basin, Paper Towel Dispenser, Soap Dispencer, General Waste Bin
- AC Outlet
- Light Switches
- Adult Scale
- Bulk Head Privacy Curtain
- Medical Equipment: Vitals Monitor, ENT Set, BP Monitor, Examination Lamps
- Desk Lamp
- DV Box
- Battery & Inverter System



STORAGE

- Sharps Container Storage
- Pull Out Refridgerated Medical Storage
- Pull Out Storage
- Dedicated Medical Storage for Scripted Medication

FRONT OF VEHICLE



Shore Plug & Shore Cable

- Situated under the front passenger seat

Figure 1.1



OEM Factory Fitted Auxillary Battery

- Located under the bonnet on the passenger side of the vehicle.
- Note : OEM primary battery is located in the Driver Cab, on the passenger side footwell.

Figure 1.2

EXTERNAL VEHICLE

Connecting the Shore Plug

- Use the 20m extension cord supplied under the passenger seat.
- Plug it into an Eskom standard wall socket, supplied via a 20A circuit breaker.
- Lift the flap on the back right side of the vehicle.
- Connect the plug securely.

**BEFORE YOU DRIVE
OFF**

Make sure to check the Shore
Plug has
been disconnected



Figure 2.1



Figure 2.2



Figure 2.3

Figure 3.1



Fresh Water Storage Refill

- Key is on the OEM ignition key set.
- Unlock the cap.
- Fill with fresh water only (max 20L).
- Lock the cap securely to prevent contamination.



Figure 3.2



Figure 3.3



Figure 3.4

Privacy Door Lock

- Allows airflow while maintaining patient privacy.
- Lift the latch inside the vehicle.
- Attach it to the back door.
- Door is secured.



Figure 4.1



Figure 4.2



Figure 4.3

INTERNAL COMPONENTS

Internal & External Stadiometer

- The internal stadiometer situated inside, on the back door measures up to 180cm. (See Fig 5.1)
- If the patient is taller than 180cm open the back door & measure against the external stadiometer (See Fig 5.3)
- The patient stands on the ground to use the external stadiometer (measures up to 210cm).
- The ground must be level for accurate external measurement.

Figure 5.1



Figure 5.2



Figure 5.3



Folding Work Station

- Tilt up the folded table top found on the side of the examination bench. (See Fig 6.2)
- Pull the tabs on either side of table top to attach & locate, then release until they click into place. (Fig 6.2)
- To close, pull the locking tabs backwards to release and push down to fold away (Fig 6.3)
- Lower the table back to the stowed position.

Figure 6.1



Figure 6.2



Figure 6.3



NB: Caution - Do Not sit on the folding table when deployed

WORK STATION



Figure 7.1



Figure 7.2

General Waste Bin

- Removeable waste bin found under the wash basin

Figure 8.1



Figure 8.2



Figure 8.3



Wash Basin

- Hands free touch activated tap



Figure 9.1



Figure 9.2

- Wave hand in front of sensor to activate the water pump.
- The pump is set to run for a maximum of 7 seconds

Soap & Paper Dispenser

- Insert and turn the key, then remove it.
- Push the button down to open dispenser and replace products.
- Close the lid until it clicks.
- Use the same key for both dispensers.
- Lock once done.

Figure 10.1



Figure 10.2



Figure 10.3



Figure 10.4

AC OUTLET

Figure 11.1


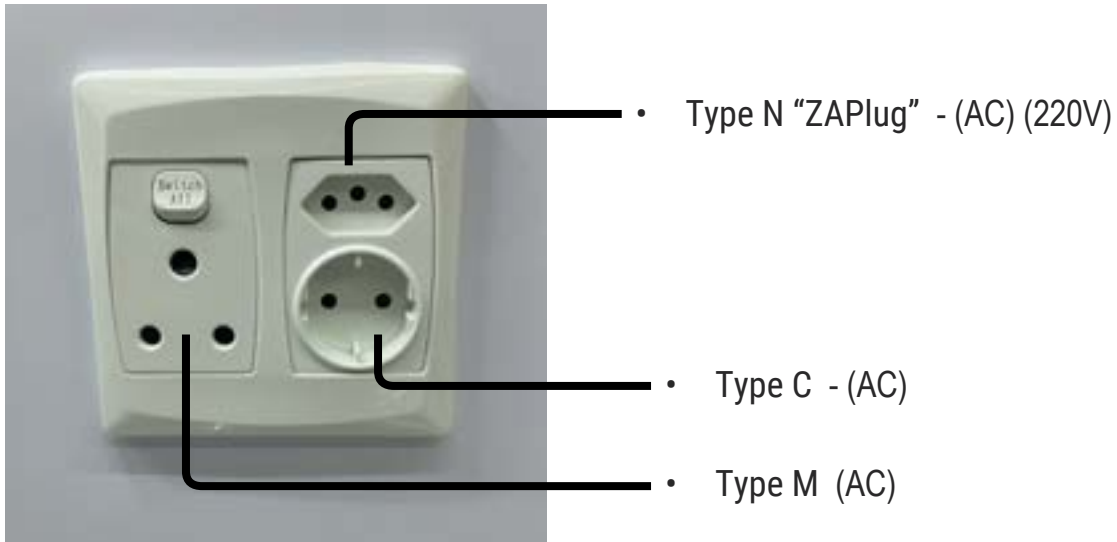


Figure 11.2

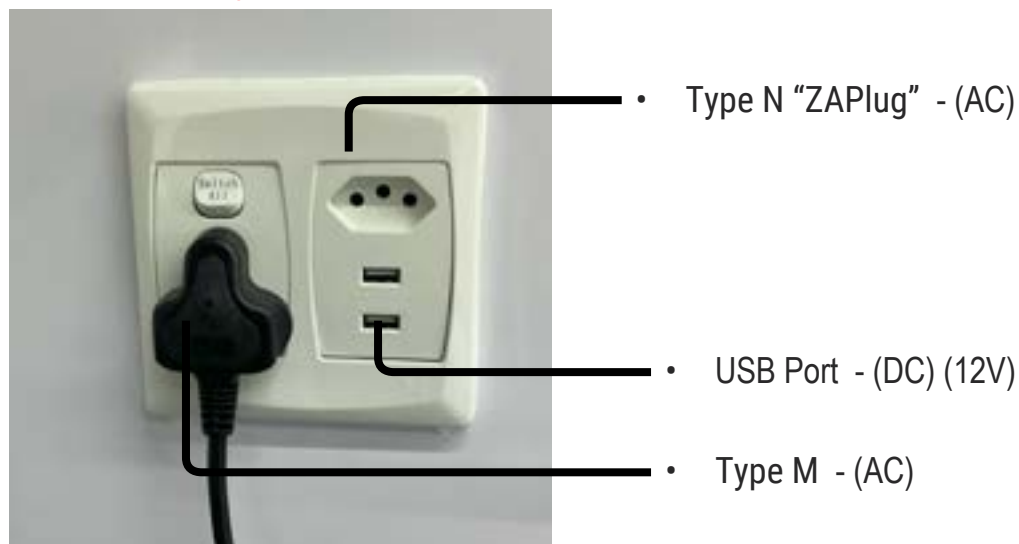

Portable Oxygen Storage

- Turn thumb fasteners to secure the portable O2 bottles in place



Figure 11.3